


OSTEOTRON IV QUICK SET-UP GUIDE

(This guide is intended as a quick reference once the user manual has been read in full)

1. Your device will come with rechargeable batteries and a charger. Place the batteries into the back of the device.
2. If using only one probe, ensure it is channel 1.
3. Insert the probe into the blue probe retainer. (If the probe is to be placed on a flat area like the collarbone or spine, you may need to tape it into place instead of using the retainer. Use the black retainer if you have a cast.)
4. Apply the ultrasound gel to the probe.
5. Attach the probe retainer to the treatment area (skin) according to physician's instruction.
6. Press and hold the power switch (for approx 1.5 sec)
7. The device sounds a series of beeps and then the treatment begins with the timer counting-down.
8. If the probe does not have full contact with the skin the treatment won't begin. Wiggle the probe to get better contact, apply more gel if required, and turn on and off again.
9. Standard settings are 30mW/cm² for 20 minutes. If the device displays something different, please contact Medi-Rent or your surgeon.



Two probe leads inserted

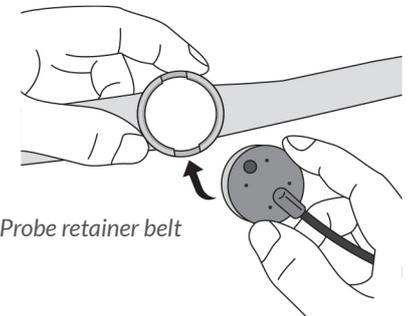

Probe retainer belt

- Your screen will look like this if one probe is used:
10. Your screen will look like this if two probes are in use:
 11. To pause, press the stop button
 12. To resume, press the stop button again
 13. When the treatment time reaches 0 the buzzer will sound and treatment will cease.
 14. Make sure the device is turned off, remove the probe from the treatment area and wipe the gel from the probe. Recharge batteries when required.

